



PrimeSafe
Primary Infection Control

ALCOHOL SURFACE SPRAY

Quick-drying surface disinfectant

ACTIVE INGREDIENTS

PrimeSafe Alcohol Surface Spray contains
70% v/v Isopropylol; 30% Inert Ingredients



USER FRIENDLY

The use of a high grade alcohol ensures minimal lingering odour after use.

STABLE

The use of a high grade alcohol ensures that the product is stable in the long-term.

EFFICACY FIRST

The use of a high grade alcohol ensures maximum efficacy from an alcohol-based product.

SAFE

Non-corrosive, non-toxic, non-staining, non-caustic and biodegradable.

CONVENIENT

Disinfection in the time it takes for the surface to dry - 1 minute.

VERSATILE

Alcohol-based products can be used on surfaces which should not be wet.

INSTRUCTIONS FOR USE

Thoroughly pre-clean and rinse the surface wherever possible. Spray PrimeSafe Alcohol Surface Spray uniformly over the surface from a distance of about 30cm away, ensuring that the surface is completely covered. Allow the surface to dry naturally, or for a minimum contact time of 1 minute, prior to using the surface again.

RECOMMENDED USES

Clinical Surface disinfection.
Laboratory surface and equipment disinfection.
Disinfection of items where a water-based product cannot be used.

FIRST AID



In case of eye contact: Flush eyes thoroughly with water. Redness may occur, but should dissipate within 60 minutes. Should it persist, consult your doctor.

In case of ingestion: Do not induce vomiting, rinse out mouth then drink a large glass of water, and consult your doctor.

In case of skin contact: No action necessary.

In case of inhalation: Remove patient to fresh air for 15 minutes.

PPE REQUIREMENTS

Latex gloves for sensitive skin, however OHSA requires the use of eye protection & good ventilation when mixing, handling or discarding chemical disinfectants.

PRODUCT DESCRIPTION

Available sizes:

20L, 5L, 1L Trigger Spray

Dilution Factor: Ready to use

Physical Form: Translucent liquid

Odour: Mild alcohol odour

pH: 6.5-7.5

Risk Classification: B / IIa

GMDN Code: 47631

GMDN Descriptor:

Medical Device Disinfectant

NRCS Reg. no.:

ACT5GNR529/242613/090/0781

Medical Device Reg. No.:

Awaiting Registration Call-up

Incompatibility:

Do not mix with other chemicals

PRODUCT TESTING

	Tested To:
Bactericidal Efficacy	SANS 51276 (EN 1276) SANS 53727 (EN 13727)
Fungicidal & Yeasticidal Efficacy	SANS 51650 (EN 1650) - Pending SANS 53624 (EN 13624)

